Artikel | Item | Article | Articolo | Artículo  
Artikkel | Výrobek | Tuote | Výrobok | Wyrób  
продукт | Ürün

4691.1.00

4691GAz\_2304

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## 1 | Delivery

- 1 x Heated back support belt
- 1 x Plug-in control panel
- 1 x Instruction manual

Retain the manual for further use and forward it to other users!

## 2 | Intended purpose

By external pressure (compression), the abdominal and back muscles should be activated and supported. Supports the spine and promotes an upright posture.

- Disc problems
- Pain of sciatica
- Lumbago
- Pain from overstrain

The additional heating effect can relieve menstrual cramps, for example.

For private use only.

## ⚠ 3 | Contraindications / Not suitable:

In the case of the following medical conditions, it is only recommended to wear this article after consultation with your doctor:

- Skin diseases/injuries in the affected part of the body, especially in inflammatory symptoms such as swelling, redness and overheating
- Allergic reactions to the material
- Sensory and movement disorders, e.g. in the case of diabetes
- Open wounds or internal injuries
- For small children or persons who can not be noticeable when the product sits uncomfortably
- Patients with implanted electronic devices, such as pacemakers or cardiac defibrillators.
- Persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and/or knowledge, unless they are supervised by a person responsible for their safety, or have been given instructions by him/her on how to use the appliance.
- If you are taking certain medications, the skin may be more sensitive to heat.

## ⚠ 4 | Notes on safe use of the product

Side effects that affect the entire organism are not yet known. Proper use is assumed. Product liability is excluded in the event of improper use.

- To prevent overheating, do not use the product in combination with a heat-generating object or heat-reflective clothing.
- If the bandage is too tight or uncomfortable, remove it immediately!
- The appliance can be used by children over 3 years old and under 8 years old under supervision, but the control must always be set to the minimum temperature value.
- Used by the times by private users.
- If your symptoms worsen or you notice unusual changes, consult your doctor immediately.
- Do not use while sleeping

## ⚠ Caution!

- Examine the appliance frequently. If there is any damage visible If the supply cord is damaged, it must be replaced by the manufacturer or a qualified person, to avoid a hazard.
- Do not pierce the surface of the product with needles or other sharp objects.
- Do not fold the item, this can lead to overheating. For safety reasons, the heating function on the back holder and the selector switch is permanently deactivated.
- Do not place heavy and/or angular objects e.g. suitcases or similar on the article, which may damage or bend the cables inside.
- To avoid damaging the power cord, make sure that it is not twisted or kinked.
- If the power cord is damaged, it must be replaced by the manufacturer or a qualified person to avoid a hazard.

- Do not bring the connection cable and switch into contact with moisture.
- The control panel becomes warm and must therefore not be covered, so that it could overheat.
- Do not use the device if any parts of it are damaged.
- Unplug the power cord from the wall outlet after use.
- Do not place other heating devices or heat generating objects on the product when it is on.
- Do not use the unit in bed.
- The cable and control unit of the device may cause strangulation and become trip hazards if improperly placed. The user must ensure that the cables are arranged in a safe manner.

### Warnings!

Observe the following instructions to avoid electric shock and to reduce the risk of burns, fire or injury:

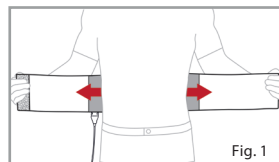
- Unplug the power cord for cleaning, or when not in use.
- Connect the power plug only to AC power with line voltages according to the rating plate on the unit.
- Do not use in the bathroom or other rooms with high humidity.

## 5 | Application

### Preparing the heatable back support belt:

If the device has been stored in very cold areas, first warm the device to room temperature before use.

- Connect the connector plug of the control unit to the support belt.
- Insert the power plug of the control unit into the socket.
- The heated back support belt is now electrically ready for operation.


### Put on the heatable back support belt:

**Fig. 1:** Hold the support belt with the hook and loop closure (inside) in the right hand and guide it around the back. Pull the two straps sideways away from the body.

**Fig. 2:** First fold in the left side and then the right one.

**Fig. 3:** Then close the hook and loop fastener at any point of the support belt.

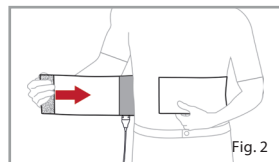*Recommendation: wear over underwear.*

### Handling

- By pressing the selector switch on the control unit, each press switches to the next heating level. The LEDs indicate the selected level.
- After pressing the selector switch in the highest heat level, the unit is switched off. All LEDs are off.

### Automatic switch-off

120 minutes after switching on, the heatable back support belt switches off automatically. If it is to be used again, the process can be restarted by pressing the selection button. The heat is then available again for 120 minutes of use.

## 6 | Cleaning

- Disconnect the power plug and ensure that the device cools down.
- Remove the control unit for washing. To do this, disconnect the power cord with the control unit from the connection of the back support belt.
- Close hook and loop fasteners during washing to avoid damaging other items of laundry. Cleaning is done by household laundry, according to care instructions.
- Ensure that the back support belt is completely dry before recommissioning or storage.
- Store in a dry and clean place.

## 7 | Technical data

Model:	4691.1.00
Rated voltage:	220-240 V ~ 50 Hz
Total power approx.:	100 W

Dimensions:	130 x 19 cm
Material:	50% Polyester, 30% Elastane, 20% Polyamid

## 8 | Warranty

The guarantee lasts for 24 months from the date of purchase for manufacturers' error in material and quality. If it is used for another purpose or operated incorrectly, no liability for possible damages can be accepted. Further information at: [info@hydass.de](mailto:info@hydass.de) | 069/954061-0.

## 9 | Disposal

Electrical and electronic devices may not be disposed of with domestic waste. Consumers are obliged by law to return electrical and electronic devices at the end of their service lives to the public collecting points set up for this purpose or point of sale. Details to this are defined by the national law of the respective country. This symbol on the product, the instruction manual or the package indicates that a product is subject to these regulations. By recycling, reusing the materials or other forms of utilising old devices, you are making an important contribution to protecting our environment.

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Hydas GmbH & Co.KG
Hydas GmbH & Co. KG
c/o atrikom fulfillment GmbH