- Ⓞ Gebrauchsanweisung
- ⓄⓄ Instructions for use
- ⓄⓄ Mode d'emploi
- Ⓞ Instrucciones para el uso
- Ⓞ Istruzioni per l'uso
- ⓄⓄ Gebruiksaanwijzing

- Ⓞ Instruções de utilização
- ⓄⓄ Brugsanvisning
- ⓄⓄ Bruksanvisning
- Ⓞ Instrukcja obsługi
- ⓄⓄ Οδηγίες χρήσης
- ⓄⓄ Návod k použití

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ⒸⒸ **Foot and Leg Cleaning Set**

Variable extension handle with 4-piece brush set to facilitate cleaning of foot, leg and back areas. Ideal for seniors and people with physical handicaps in the shoulders and arms.

**Figure A: Overview**

1 + 2: elements of extension handle

Attachment 3: for cleaning the back, legs and feet. Moisten and apply shower cream.

Attachment 4: can be used on both sides. Brush for cleaning the feet (e.g. toes and soles) and pumice stone for removing horny skin. Moisten both the foot area and the pumice stone.

Attachment 5: for cleaning the spaces between the toes. Moisten and apply shower cream.

**Figure B + C + D:**

**Assembling the extension handle / disassembly**

The extension handle can be assembled in 2 different ways. To do this, put the elements 1 and 2 together as shown in fig. B respectively fig. C until the connecting elements snap into place.

Fig. B: C-shaped version, optimal handle shape for reaching back areas

Fig. C: S-shaped version, ideal for cleaning feet and legs To disassemble the extension handle, press the two side buttons simultaneously (Fig. D) by pulling carefully apart both elements.

**Figure E: Using attachments 3 and 4**

Place the attachment on the head of the extension handle as shown in fig. E until it locks in place. To remove the attachment, press the lateral button and carefully pull off the attachment (it might still be a bit difficult to handle first).

**Figure F + G: Using the attachments 5**

Place the attachment in the small opening on the head of the extension handle as shown in figure F until it locks in place. Removal: Carefully pull off the attachment by pressing the marked area (Fig. G).

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