Imported by - Importé par:


GD import (Genius Ideas & Welllys)

190 chemin de la Frayère - 06530 Peymeinade - France

www.gdimport.com

www.genius-ideas.com

www.wellys-care.com

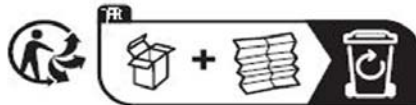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Association

ou

Magasin

ou

Déchèterie

<https://quefairemesdechets.fr>

1

2

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## GB - LIFT SEAT ASSISTANT - 014021

For correct usage, read these instructions carefully and keep in a safe place for future reference.

### User guide and warnings

1. The Lift seat assistant is to be used on stable arm chair and sofas with secure ends to hold on to when getting up.
2. The lift seat assistant is not recommended for cars, reclining chairs or any seating surface that has moving parts or that can skid. Always check the seating surface is stable and non-moving before setting up the lift seat assistant.
3. Be aware that shifting weight forward will engage the lift seat assistant.
4. Always be cautious when using this device. Prepare yourself before getting up and be careful to slowly rise out of the device until you reach a standing position.
5. If you notice that the device is malfunctioning in any way, discontinue use immediately.
6. Always set the lift seat assistant to your accurate weight setting.
7. For ages 15 and up.

### Quick start guide

**NOTE: Adjusting the body weight requires force, it is therefore recommended to request assistance for the initial setup.**

1. Place the lift seat assistant in a vertical position with the handle (which is located at the backside) showing to the top.
2. Lift the seat (the cushion / image 1) to select your appropriate body weight by moving the hydraulic cylinder along the bar (image 2). Once the weight is correctly selected, fold the Lift seat assistant to make sure the cylinder has securely snapped into the right notch.
3. Once the selection of the body weight has been effected, place the Lift seat assistant on a stable seating surface with armrests. The seating position is raised by 10 cm. Once correctly in place, the Lift seat assistant is ready for use. Sit down gently on the back part of the Lift seat assistant in order to sit down entirely.
4. To stand up, place your hands on the armrests and lean forward to activate the lifting action. It should slowly and smoothly lift you upward.

### Care and maintenance

Never submerge the Lift seat assistant in water.

If not used regularly, the lift seat assistant can become stiff and may not go down.

If this happens, set the weight at the lowest setting (50kg) and sit on it a few times until it moves freely. Adjust the weight setting before the next use and adapt it to your exact body weight.

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